


Project: INDICAID Fentanyl Rapid Detection Test	
Description: Quick Guide, US	
Document Number: PI-0162(A)	
Size: W107xH175.5	Unit: MM
Color: 4C+0C	
Paper: Wood Free 90gsm 書紙	
Last Update: 07 March 2025	

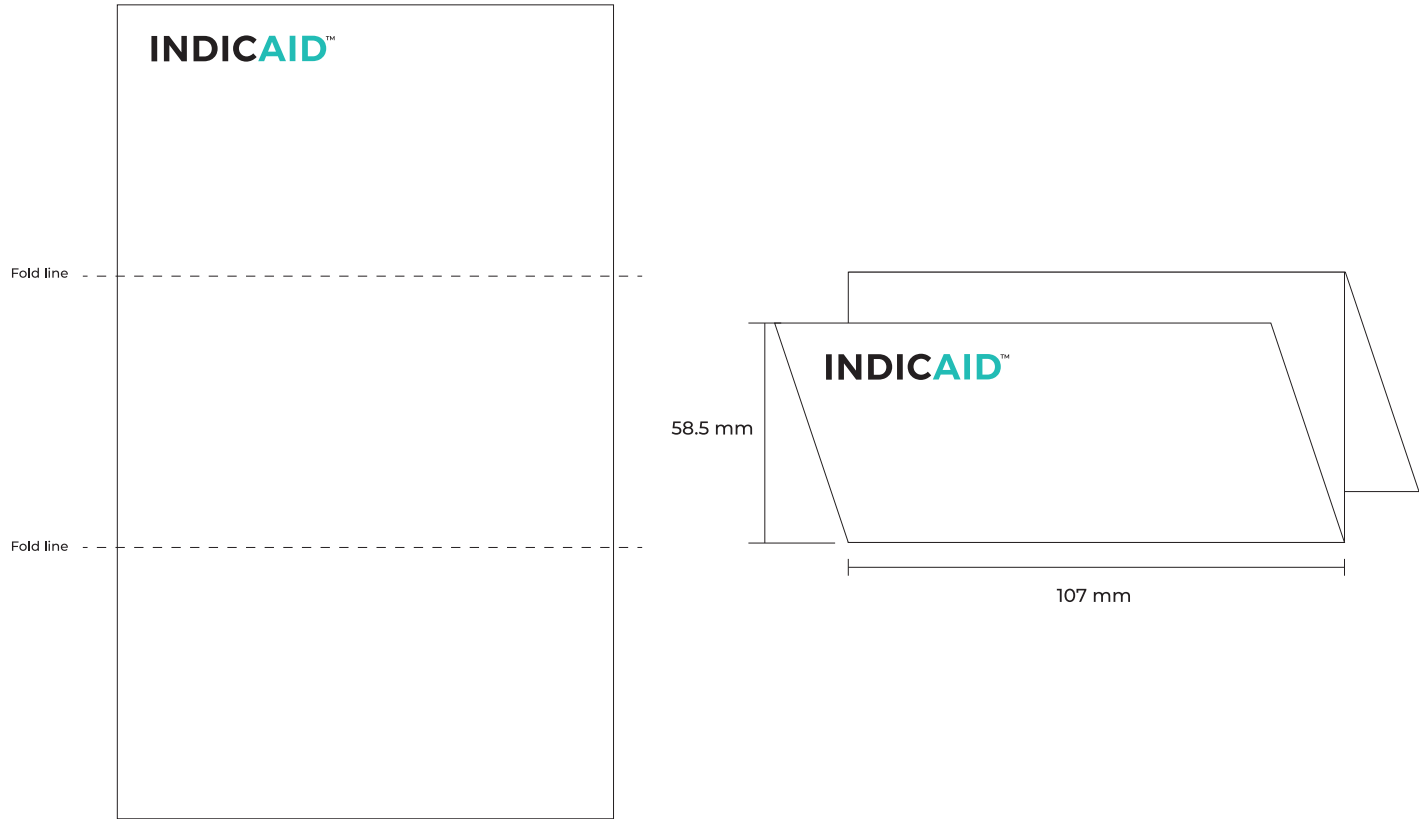

DO NOT PRINT THIS PAGE
請不要印刷此頁

..... Fold line

QUICK GUIDE

INDICAID™

Fentanyl Rapid Detection Test: A step by step guide

NOTE: A more detailed guide is included in the detection testing kit.

- 1 Dilute at least 1-10 mg powder in 5 mL water



1-10 mg
powder



5 mL
water




Fold line


- 2 Immerse the test dipstick vertically in the water specimen for at least 10 to 15 seconds.



Do not allow the level to exceed the MAX Marker Line, otherwise the test will not perform correctly.

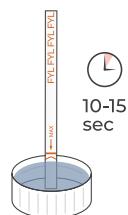

- 3 Read your results after 1 minute.

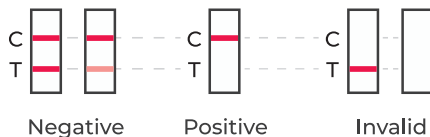

Fold line



Do not read results after 10 minutes.



Get help here

Please contact at +1 877-934-9344
(toll free number) between 9am - 5pm



PH-0162 (A) Rev Date: 2025-03-07